


## Rapid Strep Test Collection Requirements Job Aid

### Submit two throat swabs:

- 1 polyester fiber-tipped applicator (Lawson 66873). After sample collection, place swab back in paper package.
- 1 Copan eSwab (Lawson 10917). After sample collection, break off shaft at red indicator line, put swab in the provided media tube and screw cap on.

**Special Note:** During the transition to the Copan eSwab, it is acceptable to substitute the discontinued BBL Culture Swab in place of the Copan eSwab until your BBL supply is depleted.

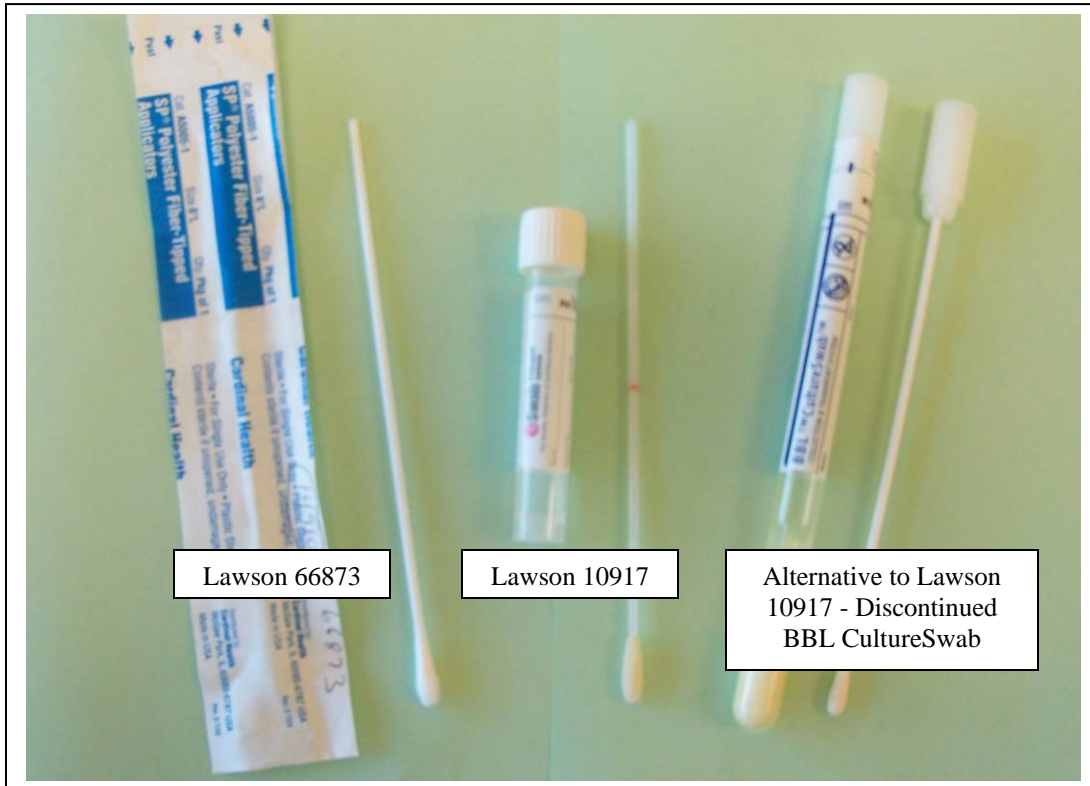

Issued 2010-10-20